




Liberator
A Prentke Romich Company
Communication without limitations

How to transfer data from the device to the NuVoice Emulation Software

Saving the data from your device to a USB stick.

- Insert your USB stick into the Accent 1000
- Press the Tools/Home Key
- Press Toolbox
- Select Transfer Memory Menu
- Select Save Entire Device Contents
- Type name for the save
- Press OK and wait
- Press OK
- Press OK
- Press GO TO HOME

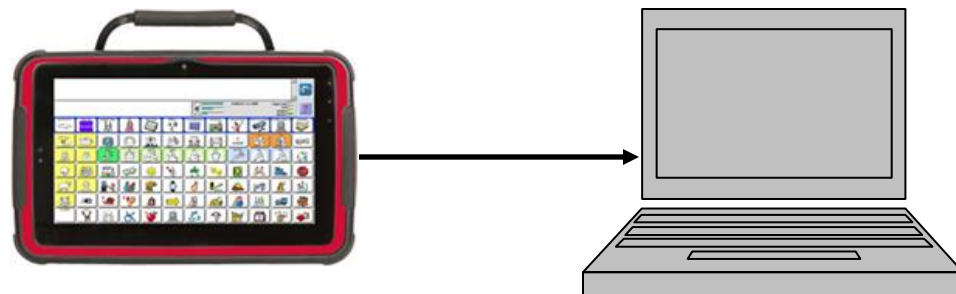

Loading the data into your computer from a USB stick.

- Insert your USB stick into the computer
- Open the Emulation Software
- Press Toolbox (F9)
- Select Transfer Memory Menu
- Select Load Entire Device Contents
- Navigate to the USB stick and choose your file.
- Press Open
- Press OK
- Press OK
- Press GO TO HOME.

You may need to choose the correct area to view the program

How to switch to the correct user area.

- Press Toolbox (F9)
- Select User Area Menu
- Select Switch User Area
- Choose the area with your data to go to
- Press OK
- Press GOTO HOME


Now you can program your device on the computer or use the Emulation Software to explore the language program.



How to transfer data from the NuVoice Emulation Software to the device

Saving the data from your computer to a USB stick.

- Insert your USB stick into the computer
- Go to Toolbox (F9)
- Select Transfer Memory Menu
- Select Save Entire Device Contents
- Navigate to the USB stick
- Type a name for the save
- Press OK and wait
- Press OK
- Press OK
- Press GOTO HOME.

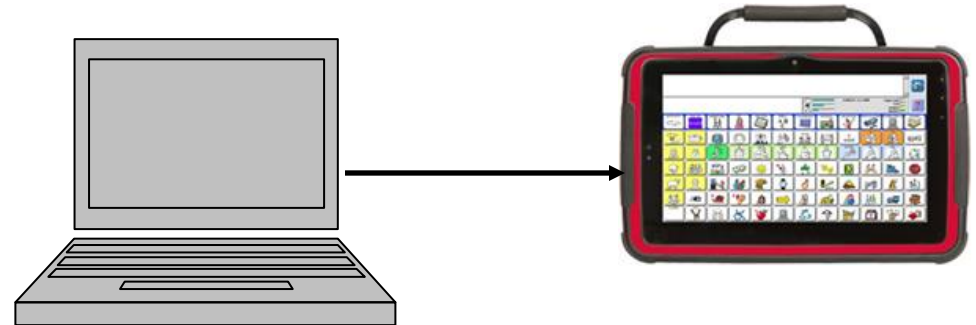

Loading the data into your device from a USB stick.

- Insert your USB stick into the Accent 1000
- Press the Tools Key
- Select Toolbox
- Select Transfer Memory Menu
- Select Load Entire Device Contents
- Choose the file to load
- Press OK
- Press OK
- Press OK
- Press GOTO HOME.

You may need to choose the correct area to view the program

How to switch to the correct user area.

- Tools Key
- Toolbox
- User Area Menu
- Switch User Area
- Choose the area with your data to go to
- Press OK
- Press GOTO HOME

Now you can view the changes made on the Emulation Software on the device.

Regional Consultants

Mark Street: 07747016660

Eleri Davies: 07827324286

Jacqui Malthouse 07825512669

Liberator contacts

Office:

01733 370470 (Option 2)