



How to transfer data from the device to the NuVoice Emulation Software

Saving the data from your device to a USB stick.


- Insert your USB stick into the Accent 1200
- Press the Tools Key
- Press Toolbox
- Select Transfer Memory Menu
- Select Save Entire Device Contents
- Type name for the save
- Press OK and wait
- Press OK
- Press OK
- Press GO TO HOME



Loading the data into your computer from a USB stick.

- Insert your USB stick into the computer
- Open the Emulation Software
- Press Toolbox (F9)
- Select Transfer Memory Menu
- Select Load Entire Device Contents
- Navigate to the USB stick and choose your file.
- Press Open
- Press OK
- Press OK
- Press GOTO CORE/GO TO HOME.

You may need to choose the correct area to view the program

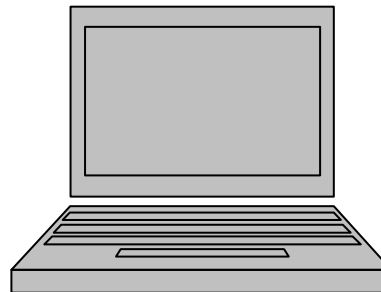

How to switch to the correct user area.

- Press Toolbox (F9)
- Select User Area Menu
- Select Switch User Area
- Choose the area to go to
- Press OK
- Press GOTO CORE/GOTO HOME

Now you can program your device on the computer or use the Emulation Software to explore the language program.




Liberator
A Prentke Romich Company
Communication without limitations

How to transfer data from the NuVoice Emulation Software to the device

Saving the data from your computer to a USB stick.

- Insert your USB stick into the computer
- Go to Toolbox (F9)
- Select Transfer Memory Menu
- Select Save Entire Device Contents
- Navigate to the USB stick
- Type a name for the save
- Press OK and wait
- Press OK
- Press OK
- Press GOTO HOME.

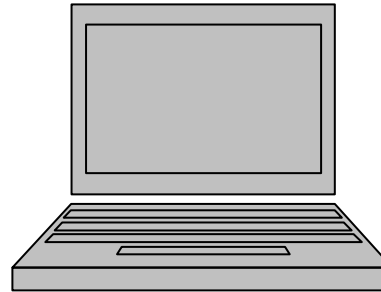

Loading the data into your device from a USB stick.

- Insert your USB stick into the Accent 1200
- Press the Tools Key
- Select Toolbox
- Select Transfer Memory Menu
- Select Load Entire Device Contents
- Choose the file to load
- Press OK
- Press OK
- Press OK
- Press GOTO HOME.

You may need to choose the correct area to view the program



How to switch to the correct user area.

- Tools Key
- Toolbox
- User Area Menu
- Switch User Area
- Choose the area to go to
- Press OK
- Press GOTO CORE/GOTO HOME

Now you can view the changes made on the Emulation Software on the device.

Regional Consultants

Mark Street: 07747016660

Eleri Davies: 07827324286

Jacqui Malthouse 07825512669

Liberator contacts

Office:

01733 370470 (Option 2)