



USING THE OFF-SCREEN KEYS WITH NuPOINT/NuEYE/USB JOYSTICK - Accent 1400/1200/1000/800

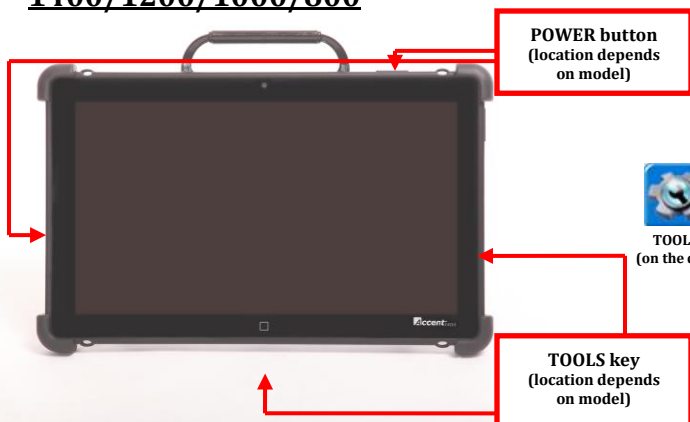

OFF SCREEN KEYS- Softkeys are present in the bottom and right side of the screen much like the softkeys that open on the left side of the screen when the tools key is activated. The softkeys display on the appropriate area when the cursor is moved to the edge of the screen at the left, right, or bottom. This eliminates the need to navigate to the Toolbox for certain Menus and/or allows you to access some 'quick' Computer MOUSE functions when using the Integrated PC on your device.

Coloured bars on each edge of the display indicates that the OFF-screen keys are ON.

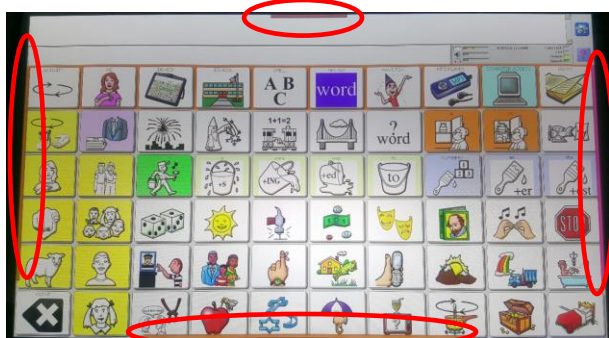

When using NuEye, looking at the desired edge of the device will display the Off-screen keys; With NuPoint, directing the cursor to the desired edge will display the Off-screen keys. Touch access users can open the softkey menus by programming an Activate Key tool set to open the desired menu.


Turning Off-SCREEN Keys ON/OFF

The Off-screen **TOOLS** key and the Off-Screen Pause key (Top-centre of display) are ON as default, but may be switched OFF if desired.

To turn ON/OFF the off-Screen Keys:

Access method **MUST** be set to NuEye or NuPoint!

1. Select the **Tools/HOME Key** then **Toolbox**.
2. Select **ACCESS METHOD MENU**.
3. Select **OFF-SCREEN KEYS SETTINGS**
4. Press **ON/OFF** to turn the keys on/off.

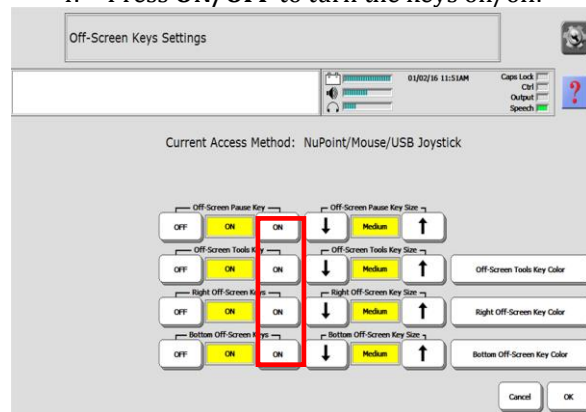


NOTE: Turning the RIGHT Off-Screen keys OFF does not stop these keys from being displayed when the TOOLS key is touched. The TOOLS key **will not** respond if you are using NuEye/NuPoint/Joystick with the cursor/pointer in this location.

Programming the Off-Screen keys

The Off-Screen keys may be programmed via the User Area Menu.

1. Select the **Tools/HOME Key** then **Toolbox**.
2. Select **USER AREA MENU**
3. Select **MODIFY USER KEYS**
4. Select the key within the block that you wish to modify – the key becomes high-lighted with a RED box
5. Select **ENTER ASSIGNMENT**

6. Type in a message if you wish or, press **INSERT TOOL** to insert a function or Menu Item
7. Change **ICON, LABEL, KEY COLOR** or **KEY SIZE** if you wish.
8. Press **OK** to Save an Exit.

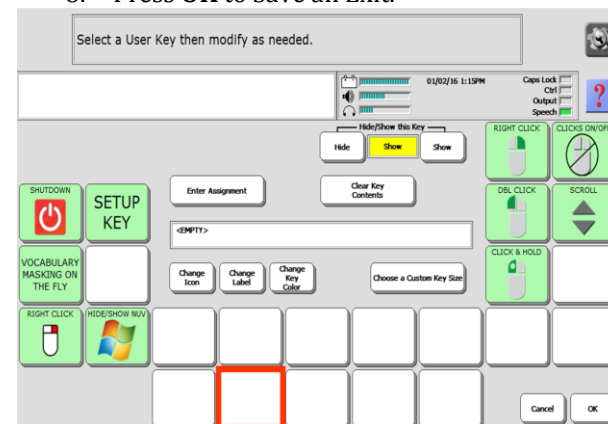

For those using Unity:

1. The LEFT Off-Screen keys (softkeys) are currently programmed with:
 - WINDOWS SHUTDOWN
 - SET-UP KEY
 - VOCABULARY MASKING ON THE FLY
 - RIGHT CLICK
 - HIDE/SHOW NuVoice
2. The RIGHT Off-Screen keys (softkeys) are currently programmed with:
 - «NEXT-MOUSE-CLICK-RIGHT»
 - «WINDOWS-CLICKS-ON/OFF»
 - «NEXT-MOUSE-CLICK-DOUBLE»
 - «SCROLL-KEYS»
 - «NEXT-MOUSE-CLICK-HOLD»

NOTE: You may edit these if you wish

Liberator contacts

Technical Support:

01733 370470 (Option 2)
info@liberator.co.uk

Address:

Liberator Ltd
Whitegates
High Street
Swinstead
NG33 4PA