

Lib14 Calibrate

Over time you may find that when you press keys, you don't see what you are supposed to see; or you may notice that the icons and images on the display screen don't seem to fit the display area as well as they once did. This is normal with a touch screen and can be easily fixed.

How to calibrate the screen.

Note: You must be in your CORE, the Toolbox, Page or Activity to recalibrate your screen. You cannot do this from a menu screen.

1. On the top front of the case, press and hold the **Toolbox key** while you activate the **#2 USER DEFINED** key. (The key that was pre-programmed as the **SET-UP** key.)
2. Release both keys.
3. You see a message that tells you to **Touch** in the middle of 4 shrinking arrows. This screen is on a time limit which requires you to touch on this spot within 10 seconds. If this screen times out you must repeat steps 1 and 2.
4. With a stylus, the eraser end of a pencil or a pen cap, **press on the dot in the middle of the shrinking arrows**. You will need to press and hold this location for 1 second. While holding this spot, you will notice the word **TOUCH** now goes to **HOLD** then to **RELEASE** when activated. Repeat this process at all 4 locations.
5. You will now see a screen asking you to **Please Do Not Touch the Screen**. This Screen will time out and return you to the Toolbox.
6. Repeat this process if needed.

Liberator contacts

Regional Consultants

Mark Street: 07747016660

Verity- Jane Hart: 07879818788

Eleri Davies: 07827324286

Jacqui Malthouse: 07825512669

Paul Whitehouse: 07879418897

Office

Technical Support: 01733 370 470

(Option 2)