



## Nu-Eye General Settings

- **Eyegaze Positioning:** this displays the position of the eyes so that the device user can be positioned correctly.
- **Tracking Status Graphics:** this turns the graphics on and off in the track status box. The graphics and the text status box help position the device correctly.
- **Pupil Markers:** this allows crosses to appear in the track status box indicating the pupils can be seen.
- **Glint Markers:** this allows crosses to appear to show where the glint is, this appears as a smaller darker cross.
- **Eye track Size:** this allows you to turn on and off the track status. If you have it on you can use **Track Status Size** to choose what size to display and its location.
- **Eye Track Size:** this allows you to determine the size of the track status box. If this is set to small it will display in the top right corner of the screen; if this is set to large it will simply display on screen in a larger box.
- **Manage Eyegaze Profiles:** this allows you to save; load or delete eyegaze profiles; useful for multiple users.
- **Calibrate Now:** this starts the calibration process (make sure you are ready).
- **Pause/Resume:** this pauses and activates the eyegaze; the device can be calibrated whilst paused.

**Software version 1.08 & above**

### Liberator contacts

#### **Regional Consultants**

Mark Street: 07747016660

Eleri Davies: 07827324286

Jacqui Malthouse: 07825512669

#### **Office**

Technical Support: 01733 370470  
(Option 2)