



Nu-Eye Pointer Settings explained

- **Highlight Keys:** this highlights the icon with a red border; it isn't possible to change the colour of the highlight it is red by default.
- **Show Cursor:** this option is to turn the cursor either on or off.
- **Cursor Size:** this allows you to change the size of the cursor used within NuVoice, the choices are small, medium or large.
- **Cursor Colour:** allows you to change the colour of the cursor.
- **Pointer Colour:** this allows you to change the colour of the pointer. The pointer is different from the cursor and is typically as circle or square.
- **Pointer Type:** this allows you to change the pointer to a circle, square or none (not having a pointer).
- **Pointer Size:** allows you to change the size of the pointer.
- **Pointer Mode:** this allows the pointer to be still or animated; if it is animated it gives the device user an idea how long to dwell.
- **Pointer Border:** this allows you to add a border of different thickness to the pointer.
- **Pointer Transparency:** this allows you to determine how transparent the pointer is, the lower the percentage the darker and less transparent the pointer is.

Liberator contacts

Regional Consultants

Mark Street: 07747016660

Eleri Davies: 07827324286

Jacqui Malthouse: 07825512669

Office

Technical Support: 01733 370470
(Option 2)