



## How to transfer data from the device to the Emulation Software (PASS).


### ***Saving the data from your device to a USB stick.***

Insert your USB stick into the Springboard Lite  
Toolbox  
Transfer Memory Menu  
Save Entire Device Contents  
Type name for the save  
Press OK and wait  
Press OK  
Press OK  
Press GOTO CORE.



### ***Loading the data into your computer from a USB stick.***

Insert your USB stick into the computer  
Open PASS  
Toolbox (F9)  
Transfer Memory Menu  
Load Entire Device Contents  
Navigate to the USB stick and choose your file.  
Press Open  
Press OK  
Press OK  
Press GOTO CORE/GOTO HOME.  
***You may need to choose the correct area to view the program***

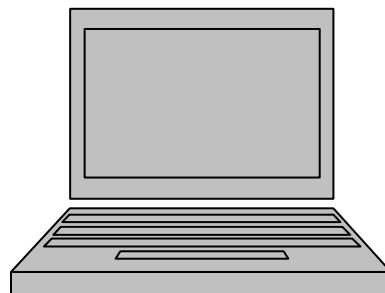

### ***How to switch to the correct user area.***

Toolbox (F9)  
User Area Menu  
Switch User Area  
Choose the area to go to  
Press OK  
Press GOTO CORE/GOTO HOME

**Now you can program your device on the computer or use the Emulation Software to explore the language program.**




**Liberator**  
A Prentke Romich Company  
Communication without limitations

## How to transfer data from the Emulation Software (PASS) to the device.

### ***Saving the data from your computer to a USB stick.***

Insert your USB stick into the computer  
Toolbox (F9)  
Transfer Memory Menu  
Save Entire Device Contents  
Navigate to the USB stick  
Type name for the save  
Press OK and wait  
Press OK  
Press OK  
Press GOTO CORE.

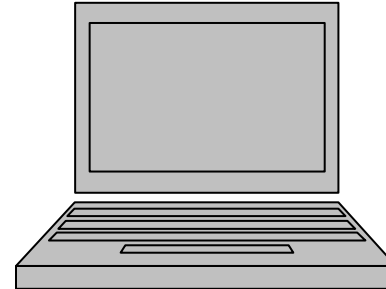

### ***Loading the data into your device from a USB stick.***

Insert your USB stick into the Springboard Lite  
Toolbox  
Transfer Memory Menu  
Load Entire Device Contents  
Choose the file to load  
Press OK  
Press OK  
Press OK  
Press GOTO CORE/GOTO HOME.  
*You may need to choose the correct area to view the program*



### ***How to switch to the correct user area.***

Toolbox  
User Area Menu  
Switch User Area  
Choose the area to go to  
Press OK  
Press GOTO CORE/GOTO HOME

**Now you can view the changes made on the Emulation Software on the device.**

### **Liberator contacts**

#### **Regional Consultants**

Mark Street: 07747016660  
Verity- Jane Hart: 07879818788  
Eleri Davies: 07827324286  
Jacqui Malthouse: 07825512669  
Paul Whitehouse: 07879418897

#### **Office**

01733 370470 (Option 2)