




# Liberator

A Prentke Romich Company  
Communication without limitations

## UK ECO™2 USB Software Update (Version 3.00 to Version 3.01)

\*You are only able to use the updater if your device is running 3.00 software and Windows 7\*

- **Read the instructions before beginning the update.**  
You may want to print this page and then proceed to the update.
- **To download and install this update you will need:**  
Your ECO2 and the ECO2 battery charger.
- The system software you are currently using in your device should be a **lower version** than the version you want to download. To check your current version, go to the Toolbox. Open the **MAINTENANCE MENU**. Look at the "System Information" box to check your Current Software Version.
- **Save your vocabulary before performing this update:**
  1. Insert your USB stick or SD card into the side of your device.
  2. Select the tools key  (F9 in PASS)
  3. Select **TRANSFER MEMORY MENU**
  4. Select **SAVE ONE USER AREA** and select the user area you would like to back up or **SAVE ENTIRE DEVICE CONTENTS**.
  5. Give the back up a name. **NAMEDMMYY**
  6. Select **OK** and wait for memory transfer to take place.

### On Your Computer

1. Go to <http://www.liberatorsupport.com/Pages/quick-reference-guides.html>
2. Click on "**ECO2**." Click on USB Update File (v3.00 to 3.01 download)
3. Download it to your **computer desktop**.
4. Plug a USB flash drive into the USB port on your computer.
5. Copy the update file to your USB flash drive.
6. Remove the USB flash drive from your computer.

### On Your ECO2

1. Plug your USB flash drive into a USB port on the right side of the ECO2.
- **Note:** ECO2 automatically performs a **Reset to Factory Defaults** during the update process. This **erases your vocabulary**. If you have made changes to your vocabulary that you want to keep, use the **TRANSFER MEMORY MENU** in the Toolbox to save the vocabulary to a USB flash drive **before** you start the update. See instructions above.
2. Turn **On** the ECO2 and plug in the battery charger.
3. Go to the Toolbox and open the **MAINTENANCE MENU**.
4. Select the **Update Software** option. Make sure the battery charger is plugged in.
5. Answer **YES** to continue.
6. Answer **YES** to continue
7. When the update is finished, remove the USB flash drive from the device.
8. If you saved vocabulary earlier, use the **TRANSFER MEMORY MENU** to load it back into the device now.
- **Note:** If you load old vocabulary, you will **lose** any MAP fixes or MAP changes that came with the new software.
9. Remove the USB drive from your device and put it in a safe place. You may also unplug the battery charger if you wish.

## Changes from ECO2 software 3.00 to version 3.01

- Improved Acapela and Natural Speech voice Pronunciations
- Minor improvements of Masking on the Fly use in core vocabulary and on pages.
- Changed Masking on the Fly tool to work only when Vocabulary Builder is turned ON.
- Enhanced Language Activity Monitor (LAM) functionality
- Repaired Random Key tool actions
- Minor Fixes and Improvements to FGPA Error, ECO.EXE Error, and others that occasionally appear.
- Improved Switch Activation and Joystick use and scanning features.
- Minor changes/corrections to Copy Pages feature in User Area Menu.
- Improved ECOpoint use when in dedicated mode with Windows 7.

### Liberator contacts

**Regional Consultants**

Mark Street: 07747016660

Verity- Jane Hart: 07879818788

Eleri Davies: 07827324286

Jacqui Malthouse: 07825512669

Paul Whitehouse: 07879418897

**Office**

Technical Support: 01733 370 470

(Option 2)