



**Liberator**  
A Prentke Romich Company  
Communication without limitations

## **Vantage Lite USB Software Update v5.08.1 to v5.10.1**

This updater can only be used if your device is running 5.08.1 system software. To check your current software version, go to the device Toolbox, open the **MAINTENANCE MENU**, and look at the "System Information" box to ensure you are running this version. If you are not, please contact Customer Support on 01733 370470 Option 2.

- ➡ This update *may* take up to three hours, so you need to prepare for this and you cannot use the device during the update.
- ➡ Read all instructions before beginning the update. You may want to print this page and then proceed to the update.
- ➡ You will need your device and the battery charger that came with the device.
- ➡ You will need to SAVE the current data (i.e. vocabulary) on your device before proceeding with the update.

### **On Your Computer**

1. Go to [http://www.liberatorsupport.com/Pages/usb-software\\_updates.html](http://www.liberatorsupport.com/Pages/usb-software_updates.html) Click on the VTL + button.
3. Under the VTL heading you will see **3 individual update files, an addition PDF file of these instructions**. Please download and read the instructions.
4. **ALL 3 files (VTLupdate, VTLupdate1, VTLupdate2)** are required for this software update and must all be downloaded to your **computer desktop first**.

➡ **NOTE: Do not download these files directly to your USB flash drive.**

4. When the files are on your desktop, plug the USB flash drive into the USB port on your computer.
5. **Copy ALL 3** of the update files from your desktop to your USB stick.
6. Eject/Remove the USB flash drive from your computer port and keep it nearby.

### **On Your Device**

1. Turn your device **ON**.
2. Plug your USB stick into the USB port on your device.
- ➡ If you have made changes to your device vocabulary that you want to keep, open the **TRANSFER MEMORY MENU** in the Toolbox and save the vocabulary to your USB stick before starting the update. Click on this link for instructions on saving data <http://www.prentrom.com/support/article/738>
2. Plug in the battery charger.
3. **Hold the ON/OFF button for 10 seconds**. The update will now begin. The device will auto update sequentially with all 3 files.
4. When the update is complete, it will ask you to remove the USB stick from the device. Unplug the USB stick.
5. Once the unit reboots and shows the core vocabulary screen, you may reload the vocabulary you saved earlier, using the **TRANSFER MEMORY MENU** to load it back into the device now.
6. Remove the USB stick from your device and put it in a safe place. You may also unplug the battery charger if you do not need it.



## Changes from Vantage Lite software 5.08.1 to 5.10.1

### Changes to the Device

- Add enhanced LAM functionality
- Modified Speech Rate that previously won't change in RealSpeak
- Defaulted Vocabulary Builder to OFF when switching to Essence - fixed
- Replace menu in VTL now matches ECO2

### Changes to the UNITY Vocabulary Programmes

- Notebook Activity hidden in all the UNITY vocabulary programmes
- Pixon sets added to UNITY in the Vocabulary Builder Menu
- Language Lab pages now included within the UNITY Vocabulary programmes
- NEWS icon updated – A new NEWS activity renamed as NOTES, the icon changed, and the instructions for use have also changed slightly.
- Updated the Calendar Year Pages that were out of date
- Corrected Dolch word list in Vocabulary Builder / Sequenced Sets
- Unity button description updated to version 5.10

### Liberator contacts

#### **Regional Consultants**

Mark Street: 07747016660

Verity- Jane Hart: 07879818788

Eleri Davies: 07827324286

Jacqui Malthouse: 07825512669

Paul Whitehouse: 07879418897

#### **Office**

Technical Support: 01733 370 470

(Option 2)