



**Liberator**  
A Prentke Romich Company  
Communication without limitations

## How to transfer data from the device to the Emulation Software (PASS).

### ***Saving the data from your device to a USB stick.***


Insert your USB stick into the Vantage Lite  
Toolbox  
Transfer Memory Menu  
Save Entire Device Contents  
Type name for the save  
Press OK and wait  
Press OK  
Press OK  
Press GOTO CORE.



### ***Loading the data into your computer from a USB stick.***

Insert your USB stick into the computer  
Open PASS  
Toolbox (F9)  
Transfer Memory Menu  
Load Entire Device Contents  
Navigate to the USB stick and choose your file.  
Press Open  
Press OK  
Press OK  
Press GOTO CORE/GOTO HOME.

***You may need to choose the correct area to view the program***

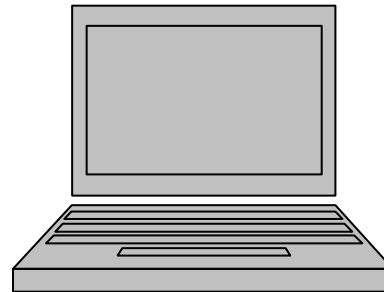

### ***How to switch to the correct user area.***

Toolbox (F9)  
User Area Menu  
Switch User Area  
Choose the area to go to  
Press OK  
Press GOTO CORE/GOTO HOME

**Now you can program your device on the computer or use the Emulation Software to explore the language program.**



**Liberator**  
 A Prentke Romich Company  
 Communication without limitations

## How to transfer data from the Emulation Software (PASS) to the device.

### ***Saving the data from your computer to a USB stick.***

Insert your USB stick into the computer

Toolbox (F9)

Transfer Memory Menu

Save Entire Device Contents

Navigate to the USB stick


Type name for the save

Press OK and wait

Press OK

Press OK

Press GOTO CORE.

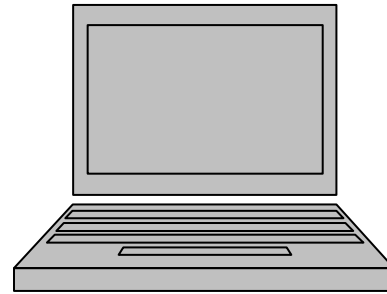

### ***Loading the data into your device from a USB stick.***

Insert your USB stick into the Vantage Lite

Toolbox

Transfer Memory Menu

Load Entire Device Contents

Choose the file to load

Press OK

Press OK

Press OK

Press GOTO CORE/GOTO HOME.

***You may need to choose the correct area to view the program***



### ***How to switch to the correct user area.***

Toolbox

User Area Menu

Switch User Area

Choose the area to go to

Press OK

Press GOTO CORE/GOTO HOME

## Now you can view the changes made on the Emulation Software on the device.

### Liberator contacts

#### Regional Consultants

Mark Street: 07747016660

Verity- Jane Hart: 07879818788

Eleri Davies: 07827324286

Jacqui Malthouse: 07825512669

Paul Whitehouse: 07879418897

#### Office

01733 3740470